


ZPad™

A Recommended Protocol

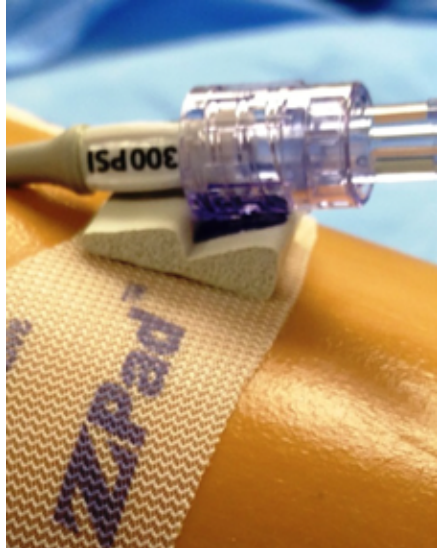

STABILIZATION DEVICE & CATHETER CUSHION

AFTER CATHETER INSERTION, APPLY SKIN PREP AND ALLOW TO DRY THOROUGHLY.


• **Remove blue cover liner**



• **Press screw (luer-lock) on blue landing pad with edge of luer-lock against ledge**




• **PEEL away liner. Place Z-Pad on skin**

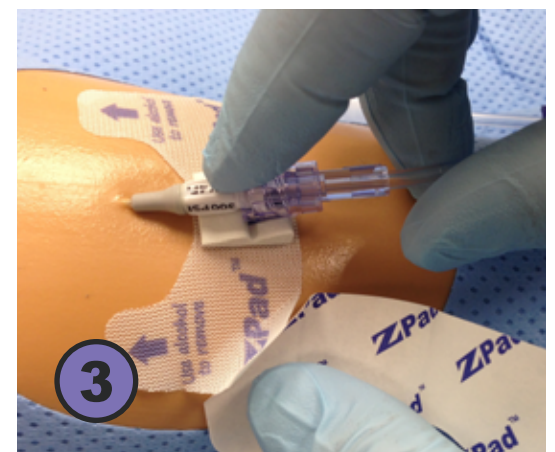

APPLY CHLORHEXIDINE PATCH AT CATHETER INSERTION SITE.

• **Apply tape over hub (Optional)**

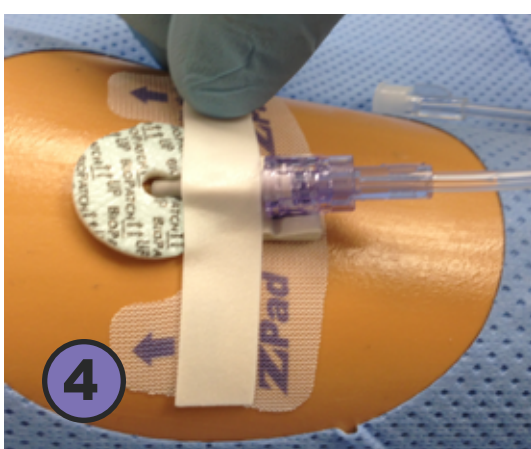

• **Chevron second tape strip (Optional)**



• **Cover with transparent dressing**



Z-Pad Removal: Change Z-pad every 7 days or sooner if indicated

A. Stretch-remove transparent dressing.

B. Stretch-remove overlying tape strips.

BE CAREFUL TO AVOID CAUSING CATHETER DISPLACEMENT

C. Use alcohol to gently remove Z-Pad

D. Separate luer-lock from Z-Pad and replace with new Z-Pad

REPLACE TUBING ACCORDING TO INSTITUTION POLICY & PROCEDURE