

ZPad™ Pediatric

A Recommended Protocol



STABILIZATION DEVICE & CATHETER CUSHION

AFTER CATHETER INSERTION, PREPARE SKIN AND ALLOW TO DRY THOROUGHLY.

- Remove blue cover liner



- Press screw (luer-lock) on blue pad with edge of luer-lock against ledge




- PEEL away liner. Place Z-Pad on skin



OPTIONAL: APPLY CHLORHEXIDINE PATCH AT CATHETER INSERTION SITE.


- Apply tape over hub (Optional)

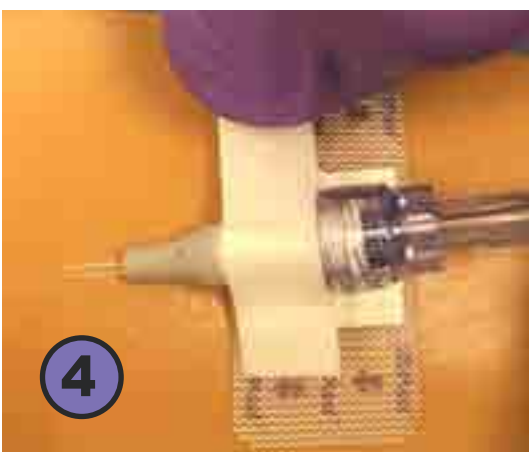

- Cover with transparent dressing



- Use hydrocolloid strip to form J-loop

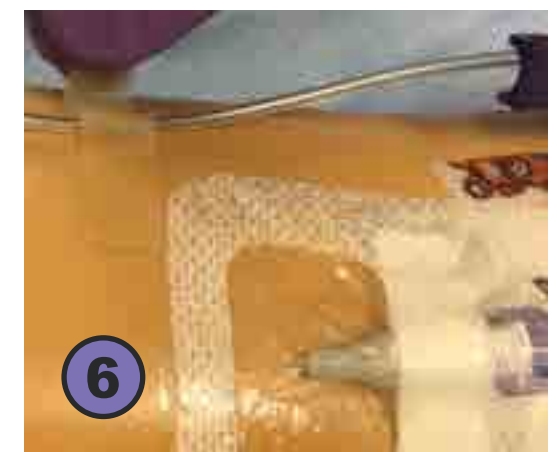

Z-Pad Removal: Change Z-pad every 7 days or sooner if indicated.

1. Stretch-remove transparent dressing
2. Roll off J-loop strip
3. Stretch-remove overlying tape strip

BE CAREFUL TO AVOID CAUSING CATHETER DISPLACEMENT

4. Gently roll Z-Pad off skin
5. Separate luer-lock from Z-Pad
6. Replace with new Z-Pad

REPLACE TUBING ACCORDING TO INSTITUTION POLICY & PROCEDURE